

# ERG 911 L ERG 911 LS

Medical couch/semi-couch safety ergometers



**SCHILLER**  
The Art of Diagnostics

# ERG 911 L AND ERG 911 LS

Medical couch ergometer and  
special ergometer for echocardiography during exercise

SCHILLER safety couch/  
semi-couch ergometer:

- ∴ Infinitely electrically  
variable and adjustable
- ∴ Low frame for easy access
- ∴ Quiet and comfortable
- ∴ Configurable



ERG 911 L: couch/semi-couch safety ergometer,  
infinitely variable from 0° – 45°

## ERG 911 L: RECLINED FOR MORE SAFETY

The couch/semi-couch safety ergometer ERG 911 L was developed for use in the cardiology sector. Special safety aspects require the use of a couch/semi-couch safety ergometer, e. g. patients with suspected problems of the cardiovascular system, during exercise following a myocardial infarction, after bypass surgery, or during special use such as cardiac catheter tests. This is when the SCHILLER couch/semi-couch safety ergometer comes into the picture. The use of a couch/semi-couch safety ergometer is also recommended for the examination of elderly or disabled patients. Thanks to the optimal positioning of the patient on the lying surface, a performance identical to bike ergometry is achieved.

## ERG 911 LS: SPECIAL ERGOMETER FOR ECHOCARDIOGRAPHY

The development of the ERG 911 LS is based on the technology of the ERG 911 L couch/semi-couch safety ergometer. Thanks to an additional pivot axis of the couch, the patient can be positioned at an angle of up to 30 degrees to the left during exercise.

This ensures a particularly favourable position for high-quality ultrasound images during stress echocardiography. The stable shoulder support ensures a comfortable and safe position of the patient while the couch is tilted to the left during echocardiography.

Apart from these special applications, the ERG 911 LS can of course still be used as a normal couch/semi-couch safety ergometer.

## ERG 911 LS: INFINITELY VARIABLE TO THE LEFT (0° – 30°)

- ❶ Opening in the lying surface at heart level for convenient ultrasound examinations
- ❷ Stable handlebar and shoulder rest for safe positioning of the patient when the couch is tilted
- ❸ Hip support for increased safety



Two operational modes:

- ❖ Remote operation with a master device (ECG, PC, ...)
- ❖ Standalone operation with either a user-definable step protocol or manual load control

Bluetooth and RS-232 connection

SpO<sub>2</sub> measurement (optional)

Stable, adjustable handlebar

Display rotatable by 180°

Elegant and easy-to-clean casing

High load precision

Very quiet (even at high speed)

Maintenance-free thanks to belt drive

Accurate blood pressure measurement unit with arm rest (optional)

Head rest and seat electrically adjustable

Opening at heart level for stress echocardiography


Infinitely electrically variable to the left (0° - 30°)

Stable steel tube construction for patients weighing up to 160 kg

Infinitely electrically variable (0° - 45°)

Large foot rest with low frame for easy access

Open construction for ergonomic ultrasound examination by the cardiologist

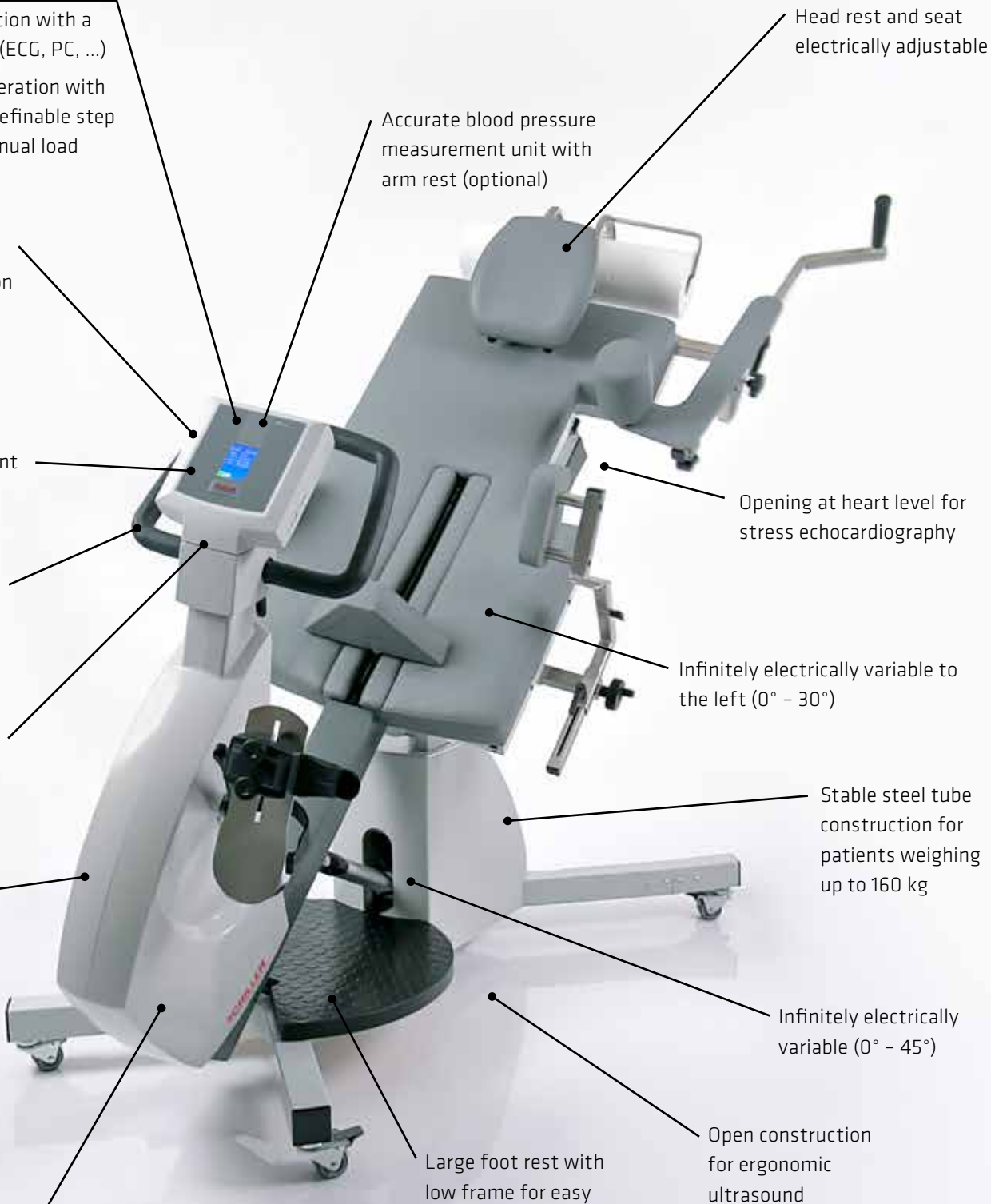



SCHILLER, founded in 1974 by Alfred E. Schiller is a successful group with 31 subsidiaries and a global sales network. Today, SCHILLER is a world-leading manufacturer and supplier of devices for cardiopulmonary diagnostics, defibrillation and patient monitoring as well as software solutions for the medical industry.

Over the past 25+ years, Schiller India has established itself as a leader in the medical technology sector. With 450+ employees, a state-of-the-art ISO 13485-certified production centre in Puducherry, and multiple R&D centres across the globe, Schiller India makes advanced healthcare equipment accessible through a network of 100 sales and service dealers across more than 45 locations. Our product range includes Critical Care, Anaesthesia, Emergency Care, Cardiology, Respiratory Diagnostics, Radiology, and Robotics.

Scan here  
to know more:



Part No. B9 100007 02/2026 Ver no. 1.1

All registered trademarks acknowledged.

**SCHILLER**  
The Art of Diagnostics

Swiss H.Q.: SCHILLER AG, Altgasse 68, P. O. Box 1052, CH -6341 Baar, Switzerland

Indian H.O.: Schiller Healthcare India Pvt. Ltd., Advance House, Makwana Road, Andheri (East), Mumbai-59. Tel.: + 91 - 22 - 61523333

No. 15/5 & 15/6, Vazhuthavur Road, Kurumbapet, Puducherry - 605 009, India

Regional Offices: Mumbai: 022-61523333 Email: sales.west@schillerindia.com Delhi: 011-41062067 / 09312432205 Email: sales.north@schillerindia.com Kolkata: 033-23593102 / 033-23593103 Email: sales.east@schillerindia.com Chennai: 044-28232648 / 044-28311021 Email: sales.south@schillerindia.com Ahmedabad: 079- 35337949 / 079- 35337950 Email: sales.west@schillerindia.com Puducherry: 0413- 220 2680 / 0413- 229 2940 Email: sales.south@schillerindia.com

CIN : U33110MH1997PTC111307

sales@schillerindia.com, help@schillerindia.com www.schillerindia.com 1800 209 8998

Follow us on : /SchillerHealthcareIndia /schiller\_healthcare\_india /schillerhealthcareindia /schiller\_healthcare\_india @schillerindia1074